

PROPHY ANGLE MAINTENANCE & CARE

POLISHER MAINTENANCE:

PROPHY ANGLE:

The Propy Angle is a precision engineered dental device. All of the gears and shaft assemblies are made of high grade stainless steel, which if cleaned and lubricate correctly will provide long, trouble-free service.

Daily Cleaning and Lubrication:

1. Remove prophy angle from straight handpiece.
2. Remove used rubber cup.
3. Remove head cap (depending on which prophy angle you have) by unscrewing the knurled nut counterclockwise or using the metal wrench provided to slide over the flange of the head cap and turn clockwise.
4. Wash the head cap and head cavity with alcohol or antiseptic soap using a toothbrush to remove any foreign particles. Dry thoroughly.
5. Lubricate the gears and head cavity by placing a drop or two of oil onto the gears of the head cap. Do not use a Q-Tip or any item with loose fibers.
6. Reassemble the prophy angle and wipe off all excess oil.
7. Confirm that the gears are meshing properly and turning smoothly by rotating the shaft at the base with your fingers. The shaft should turn easily rotating the head cap.
8. Place a new rubber cup onto the angle.
9. Slide the prophy angle down over the straight handpiece and lock it in place.



IMPORTANT: For a long dependable life, the prophy angle should be lubricated daily, if possible after each use. Keep hair away from prophy cup and head cap.

SUGGESTION: To keep hair from getting tangled in the angle, we recommend using a gentle adhesive tape around the lips, keeping hair in place away from treatment area.